


NSK Air Turbine Bearings for Dental Handpieces

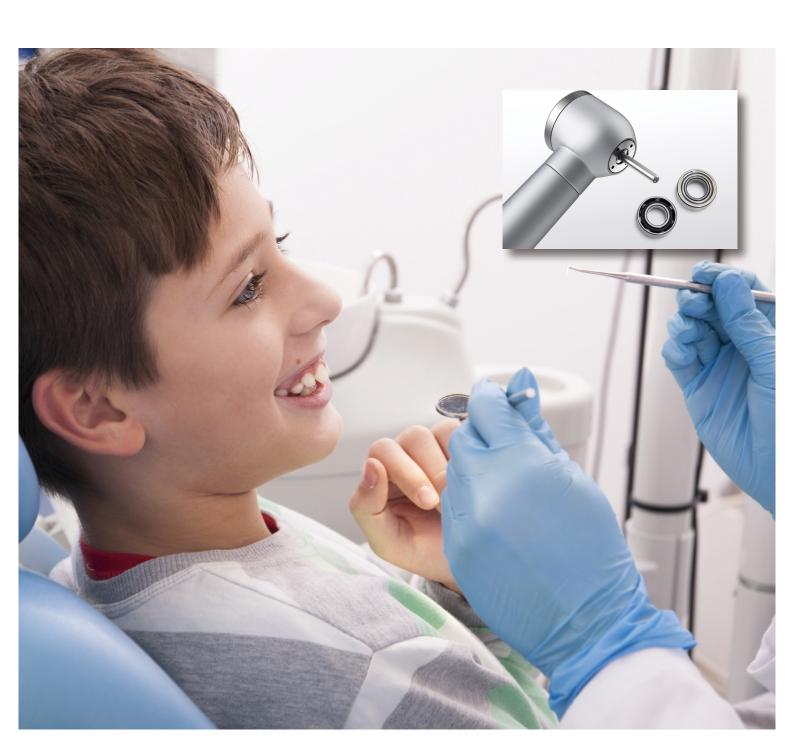






Recent Trends in Infection Control and Their Impact on Bearings

The dental handpiece is one of the most important instruments among the everyday implements used by a dental practitioner. The dental handpiece, however, poses a potential high risk of cross-contamination. To minimize cross-contamination risks and also to assure reliability and longevity of dental handpieces, these instruments are repeatedly sterilized in boiling water and disinfectants and put in an autoclave after each use. In recent years, higher awareness among patients and dental practitioners regarding cross-contamination has initiated a trend of using alkaline solutions to clean and sterilize these handpieces. These highly effective sterilizing methods could, however, cause corrosion in the bearings, which can then lead to lower performance of the dental handpieces. A corroded bearing produces increased vibration and abnormal noises in dental handpieces and eventually causes defects such as reduced rpm levels.

Solution for High Reliability and Performance, and Maximum Longevity of Handpieces

The bearings made of ES1 developed by NSK provide 10 times higher corrosion resistance than conventional bearings made of SUS440C stainless steel in an environment of exposure to a disinfectant and sterilization agent. NSK bearings offer extremely accurate revolution at an astounding speed of 400,000rpm with minimum vibration in the dental handpiece to provide safe and comfortable dental treatment.

Dental Handpiece Usage

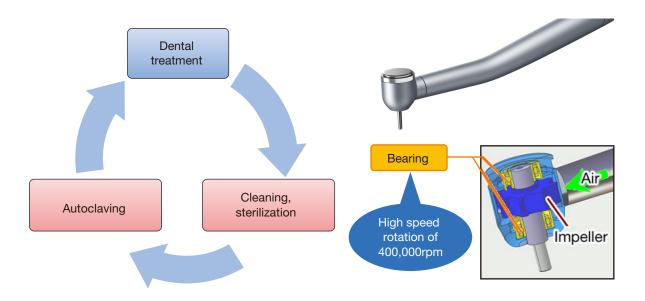

Top Runner in Bearings


Technological Strength of NSK

NSK produced the first ball bearings made in Japan in 1916. Since that time, the company has been contributing to the advancement of various industries through the use of its high-quality bearings in industrial machinery, precision machines, and automotive components. At present, the company has the largest share of the bearings market in Japan and is one of the largest bearing suppliers in the world.


For about 50 years, NSK has been developing and manufacturing bearings used in dental air turbine handpieces in Japan. In recent years, the performance requirement for bearings has become very high because of the improvement in the performance of air turbines. NSK continues to develop various technologies with the focus on ultra-high speed rotation, long life (rolling life, autoclave resistance, and corrosion resistance), low noise, low vibration, and safety through meticulous quality control and application of high-precision technologies and manufacturing technologies of bearings that the company has developed throughout its history.

For example, NSK was one of the first bearing manufacturers to discover the effectiveness of Torlon® (polyamide-imide) as a cage material and introduce it in the world. The company actively pours its energy into the development of cage materials such as polyimide and polyether ether ketone (PEEK). NSK has also developed a high corrosion resistant material called ES1* that is perfectly suitable for dental handpiece bearings. ES1 makes it possible to improve the life of bearings significantly and enable far higher quality sterilization with no cross-contamination. In a parallel effort, the company has been pushing forward with improving the precision, manufacturing method, and optimal design of cages to meet the requirements of ultra-high speed rotation performance and longer rolling life.

*Patent granted

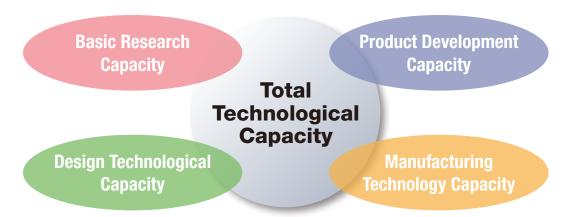


NSK's four core technologies that form the basis of its technologies and services

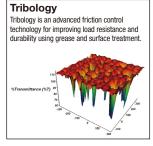

Material technology Material technology is a key to further improving the durability and reliability of bearings that are generally exposed to a severe operating environment.




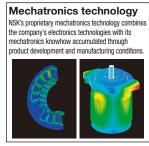




Full-fledged Global Service Network

NSK has a long track record of overseas business activities during which the company has built up a highly efficient global support network for our customers in a wide range of industries all over the world. NSK's business operation is structured around its five regional business units in the Americas, Europe/ Africa, Asia/Oceania, China and Japan. NSK operates 65 locations in Japan and 200 production sites and sales locations (excluding representative offices) as well as technical service locations in about 28 countries around the world. At all locations, our Business Division Headquarters, Functional Division Headquarters and Regional Headquarters work in close collaboration to respond fast and accurately to the wide-ranging needs of our global customers.

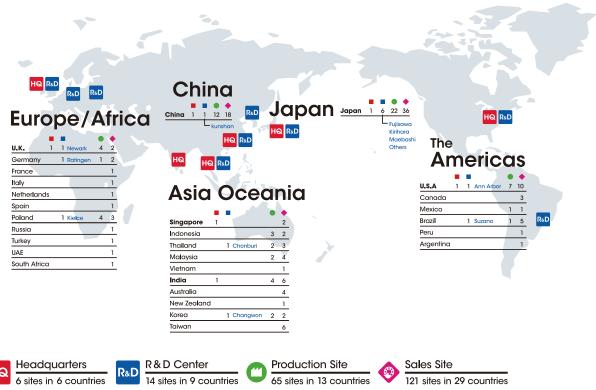

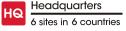

Fujisawa Technology Center (Fujisawa City, Kanagawa Prefecture)

Fujisawa Technology Center is the central hub of NSK's corporate research and development activities. Almost all business units related to the development of technologies from the product planning phase through research, development, production, and introducing the product into the market, are assembled in this Center. There are about 1,000 engineers working at the Center. It continues to produce innovative products that satisfy our customers through the combined technologies of various applications.



ISO 9001 Certified Manufacturing Plants and Wide-ranging Equipment

State-of-the-art Manufacturing Plant

NSK has two plants for manufacturing dental handpiece bearings, one in Japan and the other in Malaysia. Both plants in Japan and Malaysia are ISO 9001 certified and have the latest high-tech equipment for manufacturing high-quality miniature ball bearings. The bearings are assembled in Class 1000 (or ISO6) cleanrooms at these plants and are put through the most stringent quality management. NSK also has a system in place to assure traceability of all its products so that the company can quickly respond to any problem in product quality.



Fujisawa Plant NSK Micro Precision Co., Ltd. Kanagawa Prefecture, Japan



Malaysia Plant NSK Micro Precision(M) Sdn. Bhd. Kuala Lumpur, Malaysia

Cutting-edge Manufacturing Equipment and Inspection System

Miniature ball bearings require submicron level manufacturing precision. NSK uses technological capabilities refined over years of hard effort and production lines with the latest precision machining tools to deliver processing accuracy at the submicron level.



Multi-layer washing equipment




Assembly machines



Grinding machines



Automatic inspection equipment

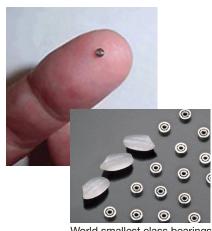

World smallest class bearings



Complete System of Measuring and Analysis Equipment

NSK has a wide-range of cutting-edge measuring and analysis equipment that is used for improving, maintaining, and confirming the manufacturing quality and developing improved dental handpiece bearings.



Roundness measuring instrument



Surface roughness/shape measuring instrument



Laser particle counter (LPC)



Fourier Transfer Infrared (FTIR) analyzer



Ion chromatography (IC) instrument



Gas chromatography (GC) instrument

Full Evaluation System


NSK has installed the company's proprietary evaluation equipment that is used to evaluate the rotation of bearings assembled in an air turbine and verify the rolling life performance. NSK has significantly improved the rolling life performance of its bearings by using ultrahigh precision balls and high-precision parts that the company has developed ahead of other manufacturers in the world.



Evaluation equipment

Quick Acoustic Analysis Software

NSK uses the company's proprietary acoustic analysis software (ACOUS-NAVI) to quickly analyze abnormal noise detected in bearings used in the products manufactured at your company and provides the required solution.

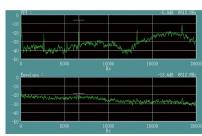


Outstanding Features of NSK Dental Handpiece Bearings

Long Life and High Corrosion Resistance ES1 Stainless Steel Bearing

NSK has developed a high corrosion resistant material called ES1 that is highly suitable for manufacturing dental handpiece bearings. ES1 has made it possible to improve the life of bearings significantly and assure highly effective sterilization with no cross-contamination. The use of ES1 has improved the bearing material, and as a result, the bearing has far superior corrosion resistance than the conventional SUS440C stainless steel bearings. The electrical and chemical verification result of our anodic polarization measurement confirms that the current density (corrosion rate) in the passive state range of ES1 bearing is about one-tenth lower than conventional stainless steel bearings, which proves that ES1 has superior corrosion resistance.


Microstructure

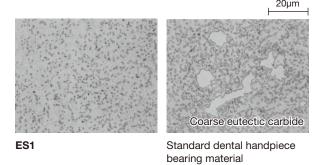

Corrosion resistance (Salt spray test)

	Specimen	Bearing
ES1		
SUS440C		

Anodic polarization measurement result

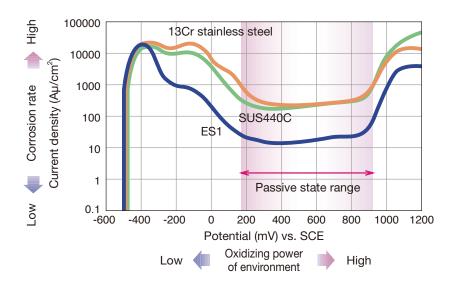






Ultra-high Speed Rotation

To maintain ultra-high speed rotation and long rolling life of air turbine bearings, it is particularly necessary to produce a high precision cage based on the optimal design. NSK makes continuous efforts to optimize the design and improve the precision of cage parts for ultra-high speed rotation. As a result, we have been successful in maintaining a stable number of revolutions even at 500,000rpm.


Examples of optimal design of elements and control values

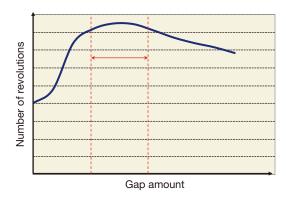

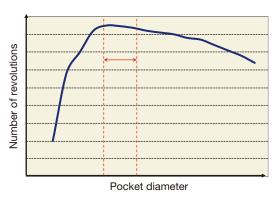

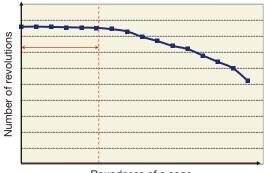

Roundness of a cage

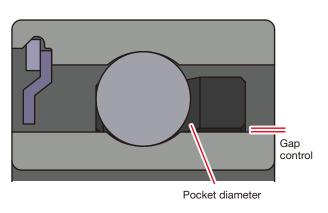





High Parts Machining Precision for Ultra-high Speed Rotation and Long Rolling Life, Low Vibration, and Low Noise


In addition to sophisticated material technologies and optimal cage designs, parts machining precision is particularly important in order to maintain the ultra-high speed rotation and long rolling life of air turbine bearings. NSK Group manufactures ultra-high precision bearing balls (far surpassing the world highest standard of balls, namely Grade 3) with original specifications. The company expertly uses these ultra-high precision bearing balls for its products. NSK has also further improved the precision of the bearing rings using the high production technologies that the company has built up over the years. The company applies these technologies for NSK air turbine bearings as standard specifications.

Example of precision of balls

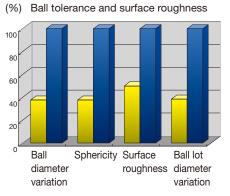

Example of ring precision

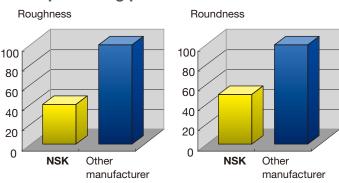

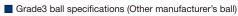

■ NSK Super precision ball specifications


Low Noise and Low Vibration

NSK has significantly minimized the acoustical noise and vibrations of bearings by improving their roughness and roundness using NSK's advanced machining technologies as well as with the adoption of ultra-high precision parts.

• Bearing noise and vibration level that surpass other manufacturers' bearings

Bearing noise (Anderon value)

100
80
40
20
NSK Other NSK Other manufacturer manufacturer

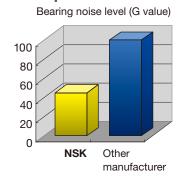



Product Lineup

Decide a code c	Width	Deep groove	Angular contact bearing		
Bearing series		bearing	Inner ring counterbore	Outer ring counterbore	
100 Series (Smooth type)	2.380mm (0.0937")				
Width		B3Z- 100X	BH3Z- 101X	BH3Z- 102X	
φ6.350mm φ3.175mm	2.779mm (0.1094")				
9 9		B3Z- 100	BH3Z- 101A	BH3Z- 102A	
150 Series (Groove type)	2.380mm (0.0937")				
Width		B3Z- 150X	BH3Z- 151X	BH3Z- 152X	
96.350mm 49.175mm	2.779mm (0.1094")				
JI		B3Z- 150	BH3Z- 151A	BH3Z- 152A	
200 Series (Flange type)	2.380mm (0.0937")				
▼		FBC3Z-200X	FBH3Z- 201X	FBH3Z- 202X	
96.350mm e9.3.175mm	2.779mm (0.1094")				
		FBC3Z- 200	FBH3Z- 201A	FBH3Z- 202A	
250 Series	2.779mm (0.1094")				
96.350nm 483.175mm		FBC3Z- 250	FBH3Z- 251A	FBH3Z- 252A	

Standard specifications

- Ceramic ball bearings
- To maximize bearing performance, NSK has adopted ABEC9(P2) grade tolerance for the inner ring bore diameter. The tolerance of other parts is ABEC7(P4) grade.
- NSK can supply bearings with two classifications of the inner ring bore diameter tolerance (from -0.0025 to -0.00125, and from -0.00125 to 0mm).
- NSK can manufacture bearings with custom laser markings upon request.
- NSK uses the high-safety low-viscosity lubricating oil CF1 as standard specification. Bearings with BF7 grease are also available.

Optional specifications


- Stainless steel ball bearings are also available.
- Bearings with different widths for the outer ring and inner ring are also available.




Integral bearing			
Inner ring c	ounterbore	Outer ring counterbore	
BH3Z- 103X		BH3Z- 104X	
BH3Z- 103A	BH3Z- 103B	BH3Z- 104A	BH3Z- 104
BH3Z- 153X		BH3Z- 154X	
BH3Z- 153A	BH3Z- 153B	BH3Z- 154A	BH3Z- 154
FBH3Z- 203X		FBH3Z- 204X	
FBH3Z- 203A	FBH3Z- 203B	FBH3Z- 204A	FBH3Z- 204
FBH3Z- 253A	FВН3Z- 253В	FBH3Z- 254A	FBH3Z- 254

Preload stabilization


Stabilization of preloading using a spring is extremely important for air turbine bearings. There are three management points ((1) to (3) below) for this task. For optimal performance of bearings, the bearing assembly height at the time of preloading is absolutely important. NSK can verify the design setting value of the bearing assembly height at your company to assure stable preloading.

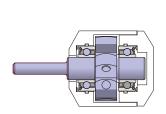

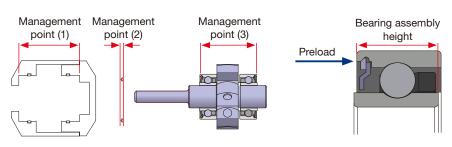

Example of Product Code

Please use the following example product code of an NSK standard dental handpiece bearing (Smooth type 100 series) as a guide to select a bearing according to requirements such as bearing width, material, and lubricant to be used.

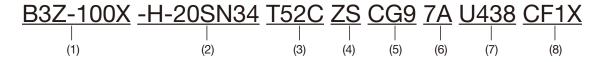

(1) B3Z-100X: Model name indicating basic bearing number for air turbine

(2) -H-20SN34: Ceramic ball (-H-26: Optional stainless steel ball)

(3) T52C: Torlon® polyamide-imide cage

C, J, P, etc.: Cage type
(4) ZS: Single shield

(5) CG9: RIC 0.008mm to 0.010mm (Recommended by NSK)

(6) 7A: Special tolerance class (ABEC7+ID: ABEC9)

(7) U438: Special specification for air turbine


(8) CF1X: Special oil (Food grade) or BF7N: Special grease

Delivering the Optimal Performance of NSK Bearings

Importance of a perfect fitting

To get optimal performance out of NSK air turbine bearings, the bearings must be correctly fitted on the shaft and in the housing. To provide this optimal performance, Class ABEC9 tolerance (from -0.0025mm to 0mm) is used for the inner ring bore of NSK bearings. NSK can supply bearings with two classifications of ring bore diameter tolerance (from -0.0025 to -0.00125, and from -0.00125 to 0mm). This choice of tolerance makes it easy to stabilize the bearing fitting on the shaft. A stable fitting reduces air turbine vibrations, noises, and irregularities in the revolutions per minute, thereby assuring long life.

Shaft material: Martensitic stainless steel (SUS400 family)
 (Please contact NSK to use bearings for shafts made of stainless steel other than martensitic stainless steel.)

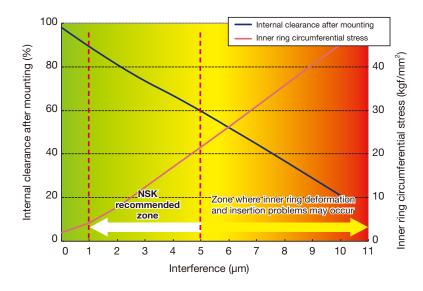



Contact NSK immediately if you encounter troubles

Bearings are used for wide-ranging applications and are a critical part that significantly affects the performance of a product and device. An air turbine in particular is a precision medical device used at an ultrahigh speed rotation of about 500,000rpm. Even a slight deviation in the performance of bearings will significantly affect the operation of a product and device. There are, however, cases where it is often difficult to determine whether a trouble symptom is caused by a bearing or some other part. NSK finds the solution to all your troubleshooting needs with its rich experience of over 50 years and the full advantage of the company's advanced analysis technologies.

Troubles you may encounter

- Case 1: Unable to get expected results despite efforts to improve air turbine performance.
- Case 2: Irregular acoustic noise (tone) of air turbine
- Case 3: The acoustic noise (low tone) is stable, but the vibrations are somewhat large and the rolling life is short.
- Case 4: The acoustic noise has a low tone that we think is highly favorable, but we are not sure.

Bearing wear

Generally, the rolling life of air turbine bearings comes to an end when the cage is worn or damaged, which slows down the revolutions per minute and causes increased vibrations. Reducing and controlling cage wear is the key factor for extending rolling life. NSK's high optimal designing capability and parts machining precision together with the customers' high assembly technologies and the dental practitioner's good maintenance of a dental handpiece are indispensable to reducing and controlling cage wear.



Please contact NSK if a cage is not evenly worn as shown in the photo above. In cases such as below, NSK may be able to help resolve the problem.

- Poor precision bearings
- Low assembly precision of handpiece manufacturer
- Poor maintenance of handpiece by dental practitioner



P: Phone \$\frac{1}{2}\$: Head Office

Worldwide Sales Offices NSK LTD. HEADQUARTERS, TOKYO JAPAN

Russia:		
NSK POLSKA SP. Z O.O.		

INDUSTRIAL MACHINERY BUSINESS DIVISION-HEADQUARTERS		NSK BEARINGS (MALAYSIA) SDN.BH	D.	NSK POLSKA SP. Z O.O.	
AUTOMOTIVE BUSINESS DIVISION-HEADQUARTERS		SHAH ALAM ☆	P: +60-3-7803-8859	SAINT-PETERSBURG	P: +7-812-332-5071
Africa		PRAI	P: +60-4-3902275	Spain:	
South Africa:		JOHOR BAHRU	P: +60-7-3546290	NSK SPAIN S.A.	
NSK SOUTH AFRICA (PTY) LTD.		IPOH	P: +60-5-2555000	BARCELONA	P: +34-93-433-5775
JOHANNESBURG	P: +27-11-458-3600	Philippines:		Turkey:	
	F. +27-11-458-3600	NSK REPRESENTATIVE OFFICE		NSK RULMANLARI ORTA DOGU TIC.	LTD. STI.
Aisia and Oceania		MANILA	P: +63-2-893-9543	ISTANBUL	P: +90-216-477-7111
Australia:		Singapore:		UAE:	
NSK AUSTRALIA PTY. LTD.		NSK INTERNATIONAL (SINGAPORE) I	PTE LTD.	NSK BEARINGS GULF TRADING CO.	
MELBOURNE ☆	P: +61-3-9765-4400	SINGAPORE	P: +65-6496-8000	DUBAI	P: +971-4-804-8200
SYDNEY	P: +61-2-8843-8100	NSK SINGAPORE (PRIVATE) LTD.		North and South America	
BRISBANE	P: +61-7-3347-2600	SINGAPORE	P: +65-6496-8000	United Sates of America:	
PERTH	P: +61-8-9256-5000	Taiwan:		NSK AMERICAS, INC. (AMERICAN HE	ADOLIARTERS)
New Zealand:		TAIWAN NSK PRECISION CO., LTD.		ANN ARBOR	P: +1-734-913-7500
NSK NEW ZEALAND LTD.		TAIPEI ☆	P: +886-2-2509-3305	NSK CORPORATION	1.+1-704-510-7500
AUCKLAND	P: +64-9-276-4992	TAICHUNG	P: +886-4-2708-3393	ANN ARBOR	P: +1-734-913-7500
China:		TAINAN	P: +886-6-505-5861	NSK PRECISION AMERICA, INC.	F. +1-754-915-7500
NSK HONG KONG LTD.		TAIWAN NSK TECHNOLOGY CO., LTI	D.	FRANKLIN &	P: +1-317-738-5000
HONG KONG ☆	P: +852-2739-9933	TAIPEI ☆	P: +886-2-2509-3305	SAN JOSE	P: +1-408-944-9400
SHENZHEN	P: +86-755-25904886	TAICHUNG	P: +886-4-2358-2945		P: +1-406-944-9400
NSK (SHANGHAI) TRADING CO., LTD.		TAINAN	P: +886-6-505-5861	NSK LATIN AMERICA, INC. MIAMI	D1 205 477 0605
JIANGSU	P: +86-512-5796-3000	Thailand:		Canada:	P: +1-305-477-0605
NSK (CHINA) INVESTMENT CO., LTD.		NSK BEARINGS (THAILAND) CO., LTD).		
JIANGSU ☆	P: +86-512-5796-3000	BANGKOK	P: +66-2320-2555	NSK CANADA INC.	P: +1-905-890-0740
BEIJING	P: +86-10-6590-8161	Vietnam:	1.100 2020 2000	TORONTO ☆	
TIAN JIN	P: +86-22-8319-5030	NSK VIETNAM CO., LTD.		MONTREAL	P: +1-514-633-1220
CHANGCHUN	P: +86-431-8898-8682	HANOI	P: +84-4-3955-0159	VANCOUVER	P: +1-877-994-6675
SHENYANG	P: +86-24-2334-2868	NSK REPRESENTATIVE OFFICE	1.104 4 0000 0100	Argentina:	
DALIAN	P: +86-411-8800-8168	HO CHI MINH CITY	P: +84-8-3822-7907	NSK ARGENTINA SRL	D 54 44 4704 5400
NANJING	P: +86-25-8472-6671		1 : +04-0-0022-7507	BUENOS AIRES	P: +54-11-4704-5100
FUZHOU	P: +86-591-8380-1030	Europe		Brazil:	
WUHAN	P: +86-27-8556-9630	United Kingdom:		NSK BRASIL LTDA.	
QINGDAO	P: +86-532-5568-3877	NSK EUROPE LTD. (EUROPEAN HEAI		SAO PAULO 🌣	P: +55-11-3269-4786
GUANGZHOU	P: +86-20-3817-7800	MAIDENHEAD	P: +44-1628-509-800	BELO HORIZONTE	P: +55-31-3274-2591
CHANGSHA	P: +86-731-8571-3100	NSK UK LTD.		JOINVILLE	P: +55-47-3422-5445
LUOYANG	P: +86-379-6069-6188	NEWARK	P: +44-1636-605-123	PORTO ALEGRE	P: +55-51-3222-1324
XI'AN	P: +86-29-8765-1896	France:		RECIFE	P: +55-81-3326-3781
CHONGQING	P: +86-23-6806-5310	NSK FRANCE S.A.S.		Peru:	
CHENGDU	P: +86-28-8528-3680	PARIS	P: +33-1-30-57-39-39	NSK PERU S.A.C.	
NSK CHINA SALES CO., LTD.		Germany:		LIMA	P: +51-1-652-3372
JIANGSU	P: +86-512-5796-3000	NSK DEUTSCHLAND GMBH		Mexico:	
India:		DUSSELDORF ☆	P: +49-2102-4810	NSK RODAMIENTOS MEXICANA, S.A	
NSK INDIA SALES CO.PVT.LTD.		STUTTGART	P: +49-711-79082-0	MEXICO CITY ☆	P: +52-55-3682-2900
CHENNAI ☆	P: +91-44-2847-9600	WOLFSBURG	P: +49-5361-27647-10	MONTERREY	P: +52-81-8000-7300
GURGAON	P: +91-124-4104-530	Italy:			
MUMBAI	P: +91-22-2838-7787	NSK ITALIA S.P.A.			
Indonesia:		MILANO	P: +39-299-5191		
PT. NSK INDONESIA		Netherlands:			
JAKARTA	P: +62-21-252-3458	NSK EUROPEAN DISTRIBUTION CEN			
Korea:		TILBURG	P: +31-13-4647647		
NSK KOREA CO., LTD.		Poland:			< A + Marrah 004 4>
SEOUL	P: +82-2-3287-0300	NSK REPRESENTATIVE OFFICE			<as 2014="" march="" of=""></as>
		WARSAW	P: +48-22-645-1525	For the latest information, please refe	er to the NSK website
				atoot illionnation, ploado fore	

Malaysia:

NSK Ltd. has a basic policy not to export any products or technology designated as controlled items by export-related laws. When exporting the products in this brochure, the laws of the exporting country must be observed. Specifications are subject to change without notice and without any obligation on the part of the manufacturer. Every care has been taken to ensure the accuracy of the data contained in this brochure, but no liability can be accepted for any loss or damage suffered through errors or omissions. We will gratefully acknowledge any additions or corrections.

For more information about NSK products, please contact:-

